

# Digital Breath Alcohol Tester

## COV14

|                               |  |
|-------------------------------|--|
| • <b>Alcohol test range::</b> | 0-0.19%BAC   |
| • <b>Accuracy:</b>            | 0.01%BAC   |
| • <b>Warm up time:</b>        | 10-15 s  |
| • <b>Response time:</b>       | less then 5s   |
| • <b>Features</b>             | <ul style="list-style-type: none"><li>• Quick response and resume capacity</li><li>• Auto power off for saving battery</li><li>• Key Chain</li><li>• 5 x Mouth Pieces</li><li>• 2 x AAA Batteries (not included)</li></ul>   |
| • <b>How to use:</b>          | <ul style="list-style-type: none"><li>• Press the "POWER" button once, the screen will display a 10 or 15 seconds count down indicating the tester has entered into "warm up" mode.</li><li>• When the "blow" symbol displayed , blow into breath tester for 3-5 seconds, then read the results from the LCD.</li><li>• The "Caution" symbol will be displayed if the alcohol concentration is between 0.02%-0.05% BAC.</li><li>• The "Danger" symbol will be displayed if the alcohol concentration is over 0.05% BAC</li></ul> |

